

# CleanCide Wipes Cleaning/Disinfecting Procedure



## To Clean:

Remove saturated wipe from canister and wipe area to be cleaned for 30 seconds or until visibly clean.

## To Disinfect:

Remove saturated wipe from canister and wipe hard, non-porous surface to be disinfected. Allow treated surface to remain visibly wet for at least 5 minutes. Allow treated surface to air dry. No rinsing or wiping is required.

EPA Reg No. 34810-36 EPA Est. No. 96006-ME-1 NSF Reg No. 158328

## Efficacy Summary Data

**CleanCide** is on EPA's List N and meets all criteria for use against SARS-CoV-2, the virus that causes COVID-19

### Bactericidal Activity in the presence of 5% blood serum is effective against:

#### Organism

**Contact Time: 5 Minutes**

**ATCC#**

Enterococcus faecalis Vancomycin Resistant (VRE)	51575
Escherichia coli O157:H7	43888
Carbapenem-resistant Klebsiella pneumoniae	BAA-1705
Listeria monocytogenes	19117
Salmonella enterica (formerly Salmonella Choleraesuis)	10708
Staphylococcus aureus	6538
Staphylococcus aureus - MRSA	33592
Staphylococcus epidermidis - MRSE	51625

### Virucidal Activity in the presence of 5% blood serum is effective against:

#### Virus

**Contact Time: 3 Minutes**

**Strain**

Hepatitis B (HBV)	NADL
[Hepatitis C (HCV) , Bovine Viral Diarrhea Virus	1059

**Contact Time: 5 Minutes**

Adenovirus Type 2	VR-846
Feline Calicivirus (surrogate for Norovirus)	F-9
Herpes Simplex Virus Type 1 & Type 2	VR-733, VR-734
Human Immunodeficiency Virus Type 1(HIV-1) (AIDS Virus)	HTLV-IIIB
Influenza A Virus	H3N2
Pandemic 2009 H1N1 Influenza A Virus	2009 H1N1
Rotavirus	WA, Univ. of Ottawa
Vaccinia Virus	VR-119